

# BD Nexiva™

## Closed IV Catheter System—Dual Port

### Catheter


- + High-pressure extension set\*
- + Stabilization platform
- + Access port
- + Access port



This document is not intended as a supplement to or replacement for the Instructions for Use. Please consult product labels and inserts for instructions for use, indications, contraindications, warnings and precautions.


\* 24 gauge catheter systems should not be used with power injectors.



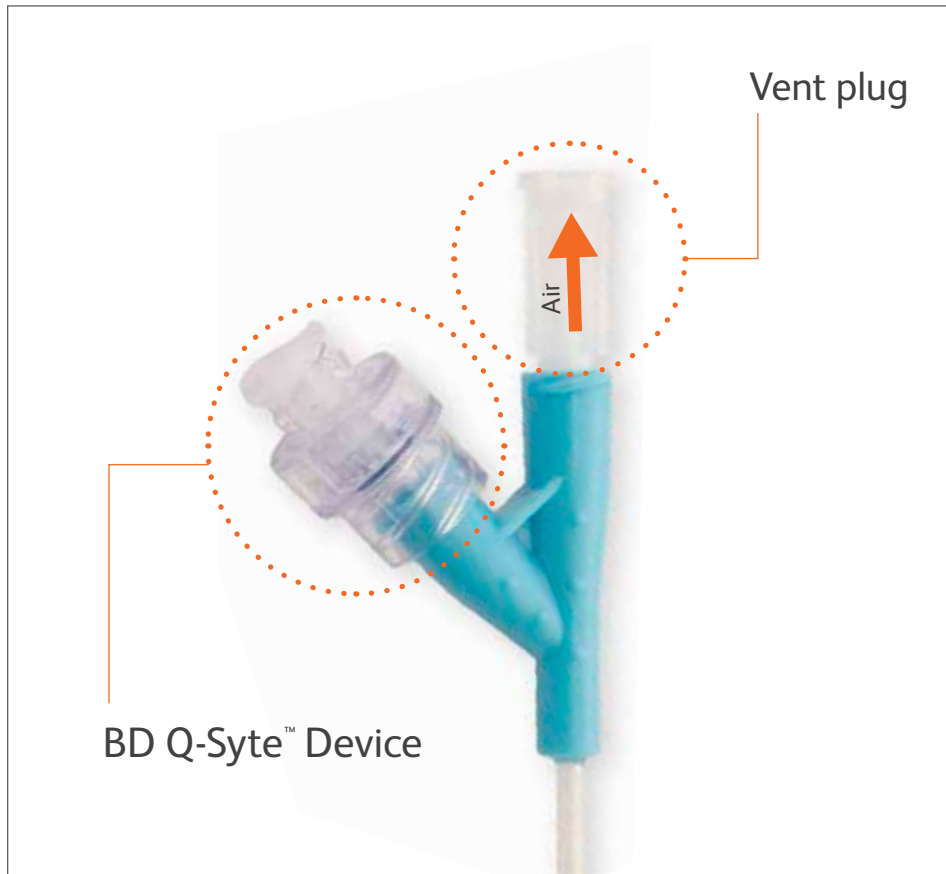


## Secure connections

- > Vent plug must remain in place during insertion for
  - Flashback
  - Blood containment
- > BD Q-Syte™ Luer access split septum

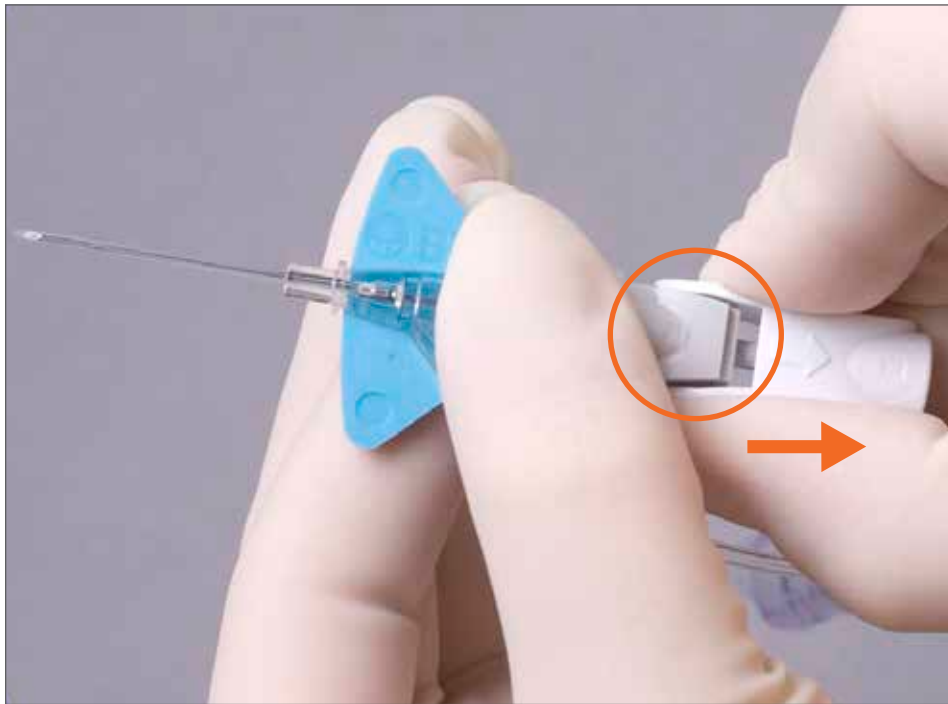

## Release seal


- > Loosens catheter for easier advancement
- > Pull back ~ 1/8 inch

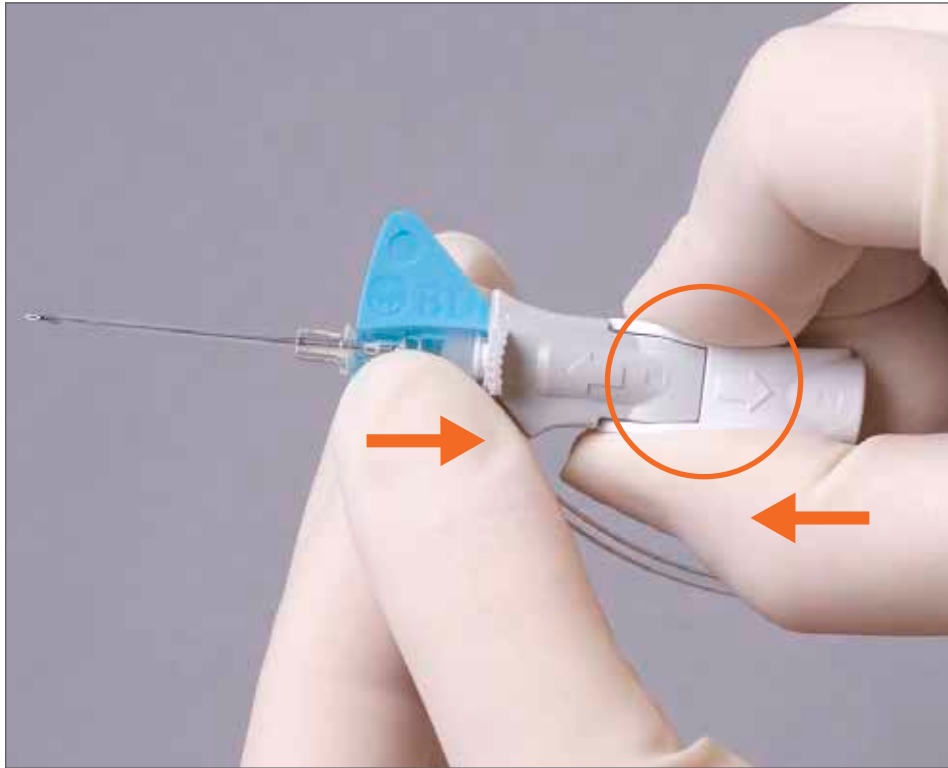


“Color to white  
everything tight”

- > Push colored platform and white grip back together


Keep it **sharp**

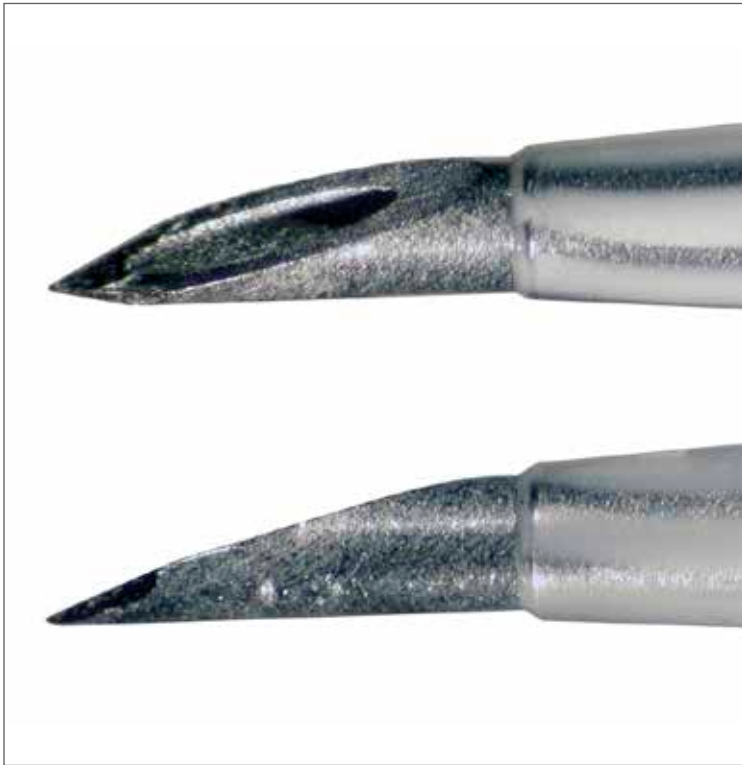

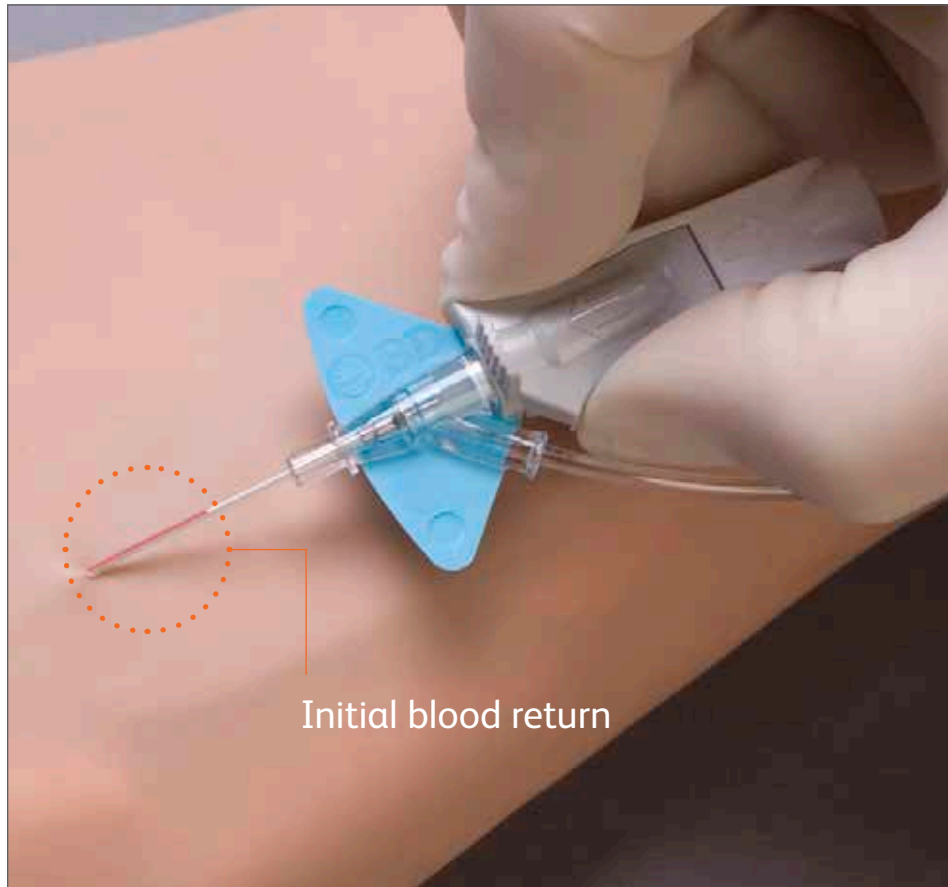

## BD Instaflash™ Needle Technology


- > Immediate blood return is seen along the catheter

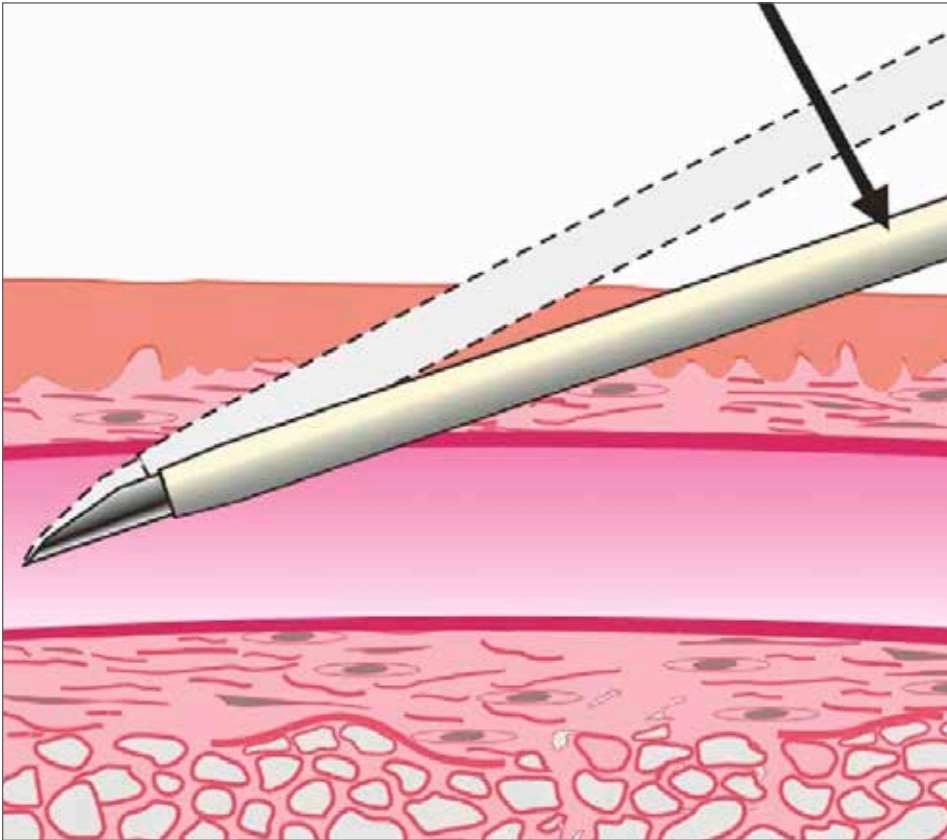

## Lower and advance

- > Lower and advance the entire catheter and needle unit slightly to ensure that the catheter tip is within the vessel

# Forward first

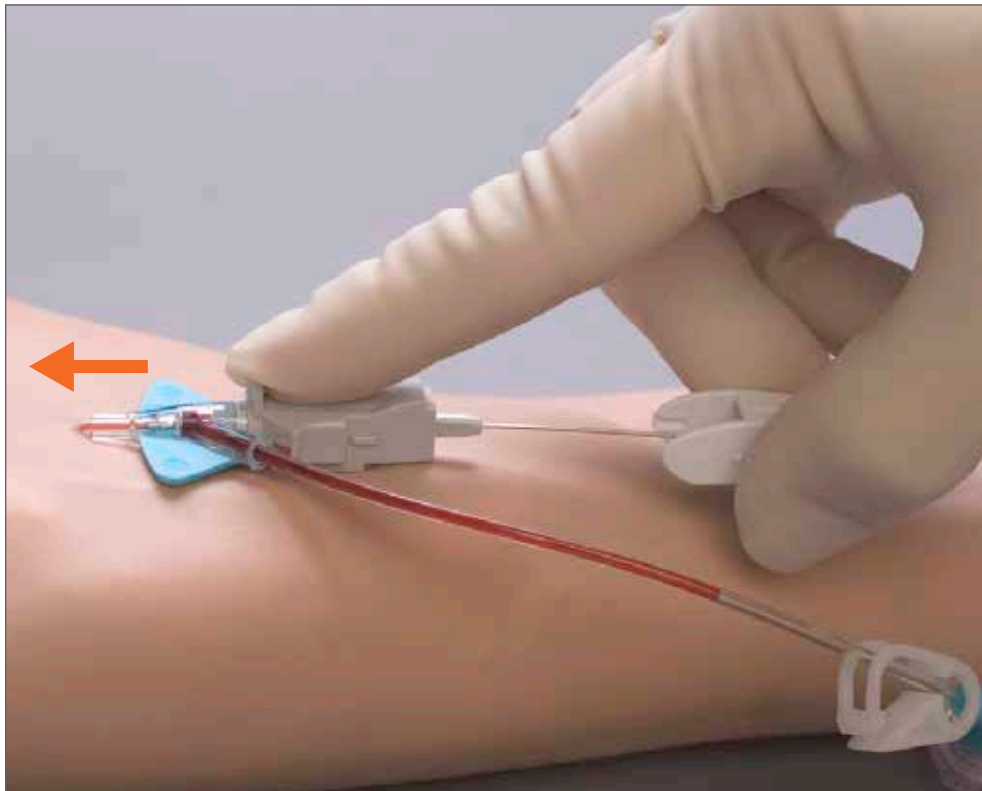



# Pad to tab





Forward first


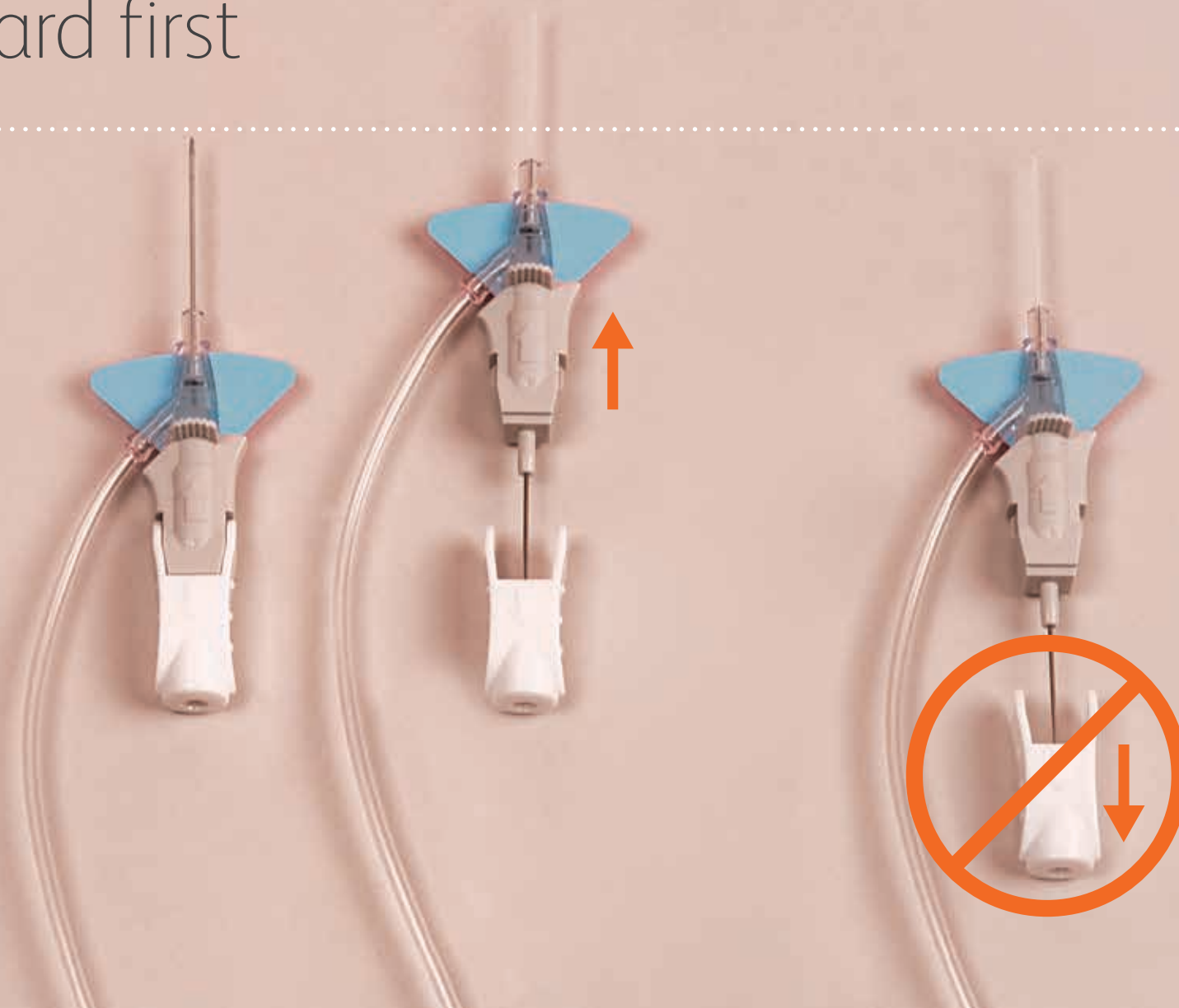


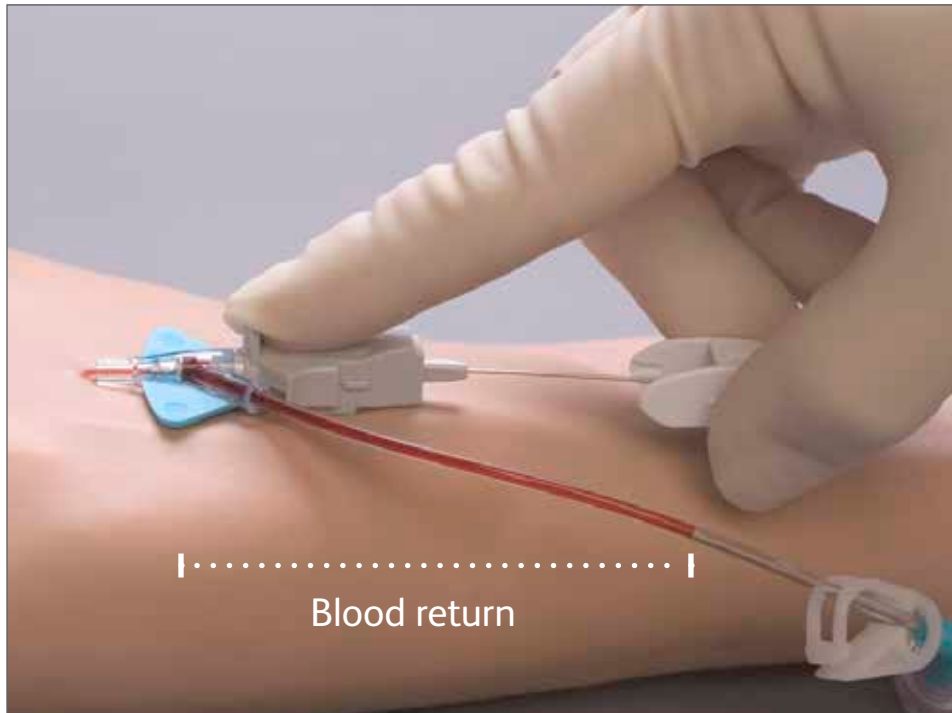


## Secondary flash

- > Blood return continues up the extension
- > Continuous flow confirms the catheter tip is within the vessel lumen
- > Quickens as needle is withdrawn


## Stabilize catheter and remove needle

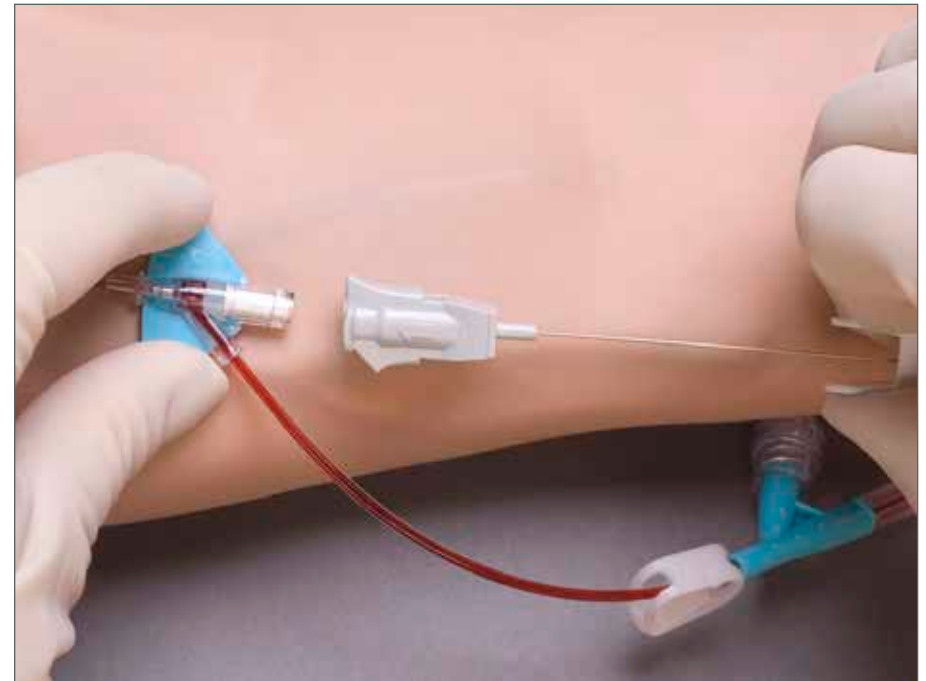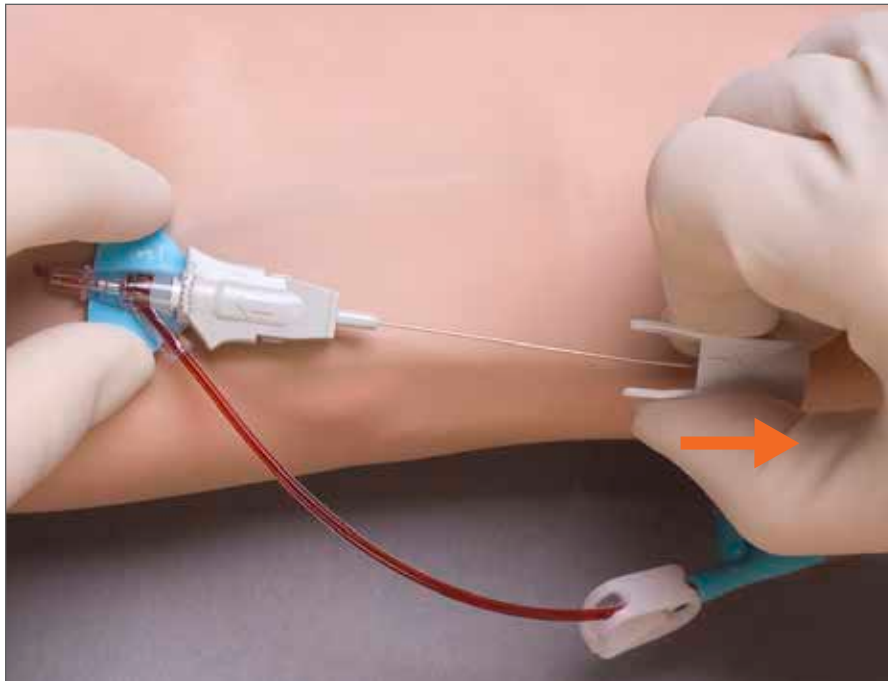

## BD Nexiva™ Closed IV Catheter System—Dual Port

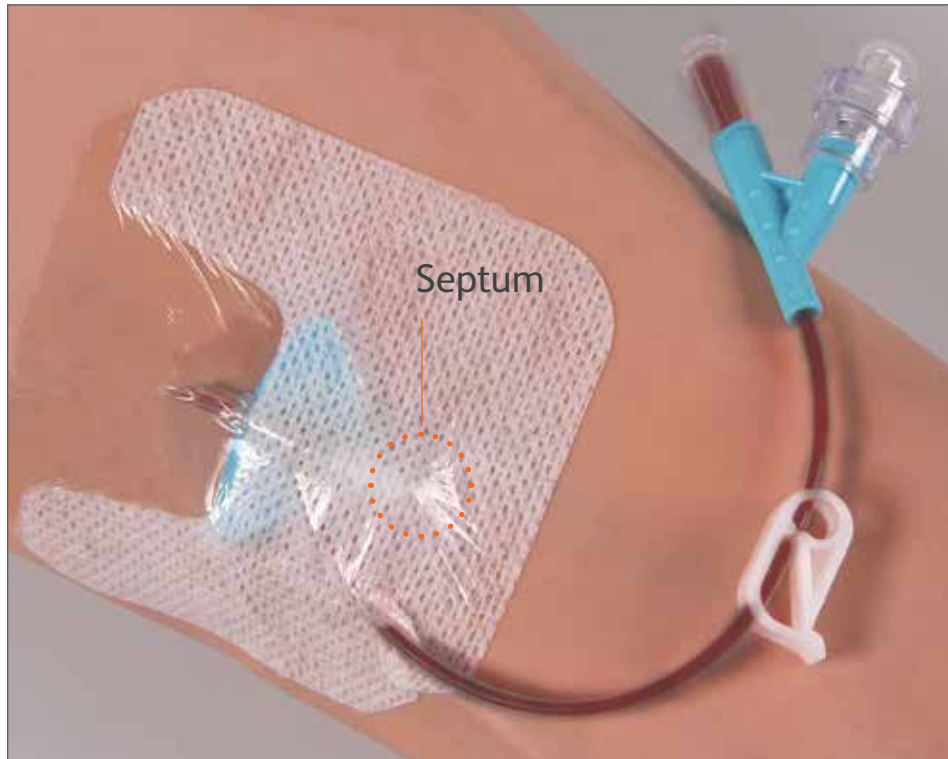


- > Secure and apply the sterile dressing

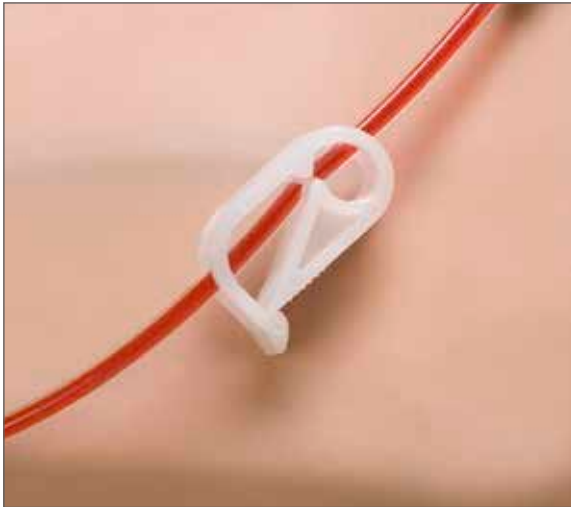

## Clamp and replace vent plug


- > Clamp extension set
  - Prevents blood leakage
- > Replace vent plug with BD Q-Syte™ Device

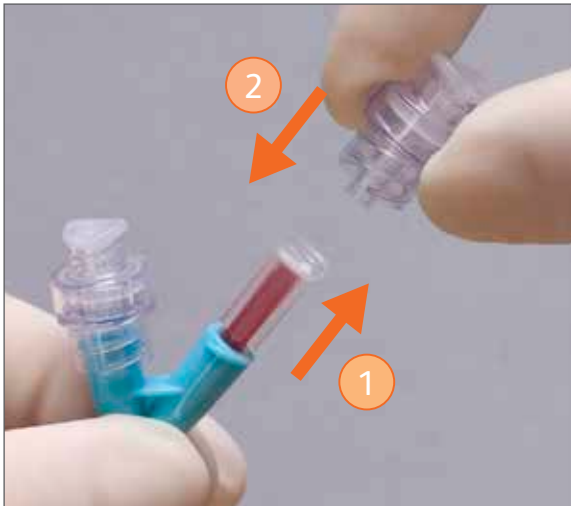



## Unclamp and flush

- > Release the clamp to flush or infuse
- > Complete securement of extension tubing
- > Single lumen catheter
  - Compatible meds only

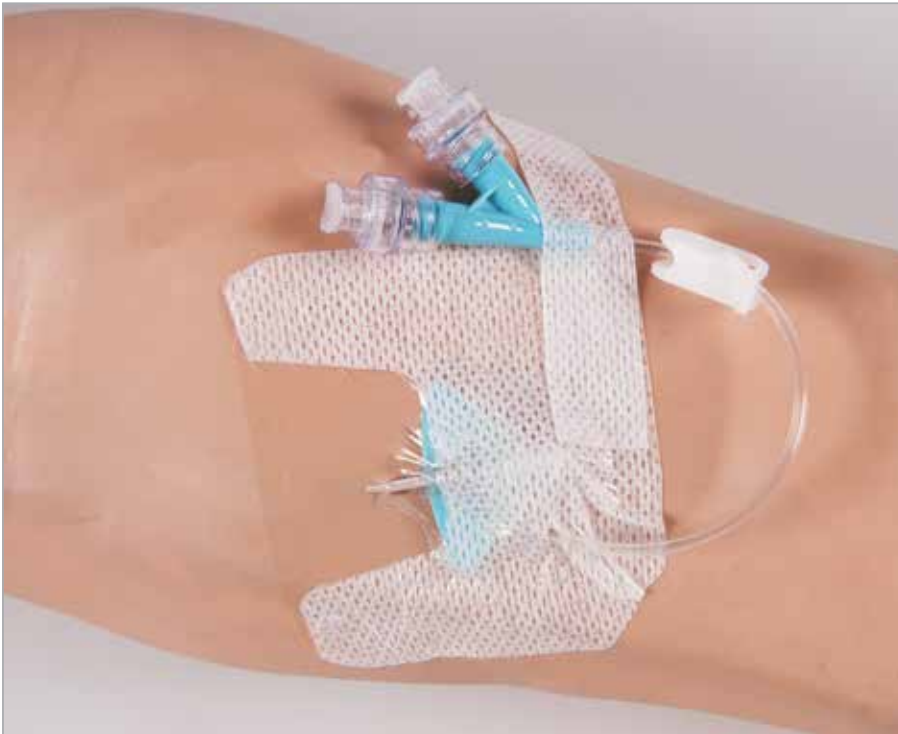

## Disconnection:

- > Clamp before disconnecting to minimize the reflux of blood
- > Hold the BD Q-Syte™ Device during disconnection and use a straight-off approach

BD, Sandy, Utah 84070  
1.888.237.2762  
[bd.com/nexiva](https://bd.com/nexiva)

**bd.com**

BD, the BD logo, Instaflash, Nexiva and Q-Syte are trademarks of Becton, Dickinson and Company or its affiliates. All other trademarks are the property of their respective owners. © 2021 BD. All rights reserved. BD-6039 (11/21)

